


Important: Please read the Drinkup Quick Start Guide & the Drinkup User Manual for comprehensive information. Risk of choking on fluids & small parts. Assessment required prior to unsupervised use by children or adults. Allow boiling/hot drinks to cool.

Advice, Customer Service & Spares: Please contact us at: Email: info@drinkup.uk.com, visit our website for further information: www.drinkup.uk.com, or contact your supplier.

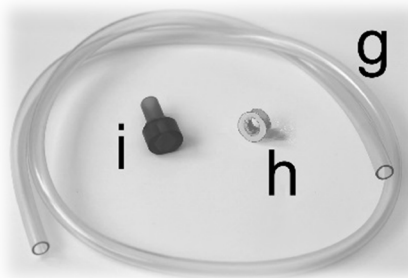


Pack contents:


- g) Drinking Tube
- h) Tube Collar
- i) Bite Valve*

*Drinkup products are only supplied with genuine CamelBak® Big Bite™ valves


Preparing the Drinkup:

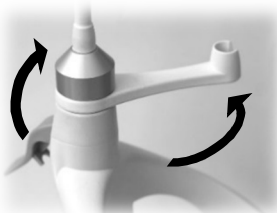


- 1) raise lever
- 2) move tube holder aside
- 3) fill jug with drink & fit lid (or use other vessel e.g. beaker, mug, can)

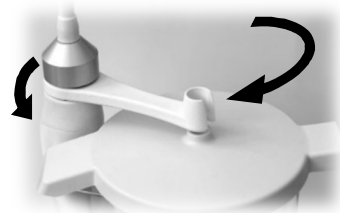


- 4) place jug into Drinkup base
- 5) return tube holder to align with hole in lid
- 6) lower lever to fully 'closed' position

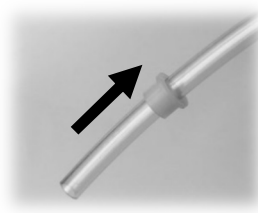


- 7) pass drinking tube through tube collar (with collar's shoulder uppermost)

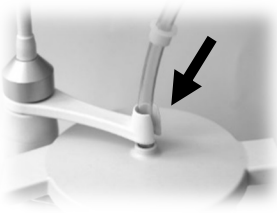


- 8) pass drinking tube through tube holder & lid until reaching jug base

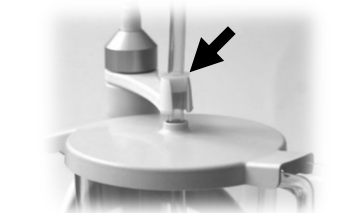


- 9) slide tube collar into tube holder up to shoulder, securing drinking tube in place

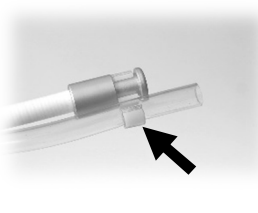


- 10) press other end of drinking tube into clip at end of flexible arm, leave 25mm overhanging

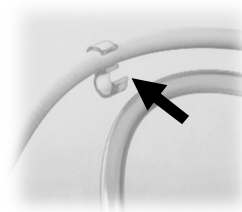


- 11) press drinking tube into other clips along flexible arm, leaving slack tube above jug

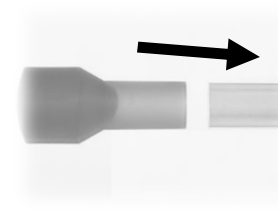

- 12) return to end of drinking tube, fit bite valve correctly

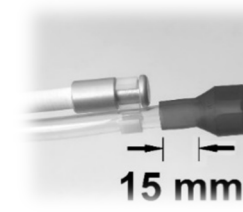

- 13) ensuring full engagement (15mm) on drinking tube and

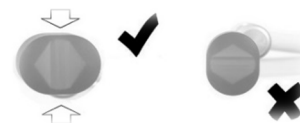

- 14) correct orientation in relation to bite valve's opening slit, adjust flexible arm to suit and check bite valve orientation

Using the Drinkup: Apply gentle pressure on bite valve with lips, teeth or gums whilst forming lip seal around bite valve so air cannot enter drinking tube, then suck as using typical drinking straw. If you cannot suck the drink along entire length of tube in one go, release the pressure applied to bite valve, which allows bite valve to close. This prevents drink returning to the vessel (or from siphoning out). Rest/take a breath. Before trying again, ensure complete lip seal is formed around bite valve, then apply light pressure to bite valve & suck as before. If airtight lip seal around bite valve is not achieved before bite valve is opened, drink will return to the vessel and the process will need to be started again. **Caution:** Avoid unwanted siphon.

'Safe-Siphon' Mode: If you cannot suck the drink along the tube in the usual way, Drinkup can be used in the 'Safe-Siphon' Mode. A carer/assistant can 'prime' the drinking tube by using a special bite valve adaptor & syringe, and the Drinkup can be set-up to take advantage of gravity and a deliberate siphon. Please see 'Safe-Siphon' Mode info sheet for full details.

Cleaning & Hygiene: Wash drinking tube & bite valve by hand in warm water & mild detergent. Sterilising solution may also be used. Thoroughly rinse & drain/drip-dry. A tube cleaning brush is available to help clean inside the drinking tube. Take care not to damage opening slit/valve faces of bite valve. The Drinkup hardware & the supplied jug/lid may be washed as above or in a dishwasher at a max temp of 85 C. Strong bleach or aggressive chemicals should not be used. Do not autoclave or place the Drinkup in microwave oven. The supplied jug is microwave safe.

To reduce risk of infection in hospitals, care homes etc. we advise frequent cleaning of the Drinkup hardware and changing/disposal of the drinking tube, tube collar and bite valve (Consumables Parts Pack). Please follow the local protocols, training or guidance which may apply at your location.



Regd. Office: Adret GIBS Ltd, McLellan Harris & Co., Waterloo Chambers, 19 Waterloo Street, Glasgow G2 6AY. Adret GIBS Ltd. reserves the right to change product design, specification, availability & price without prior notification. Designed, developed & assembled in U.K. Patent(s) apply. Issue: CPP_V2_0215. Drinkup logo™ & Adret GIBS logo™, Adret GIBS Ltd. © 1997 – 2017